



Creating DDX Cases Through Romexis

For practice to lab digital file transfer.



Select your patient

Planmecca Romexis 3.8.3.R 17/12/14 - Default Provider (sysadm) - Licensed to: Henry Schein Demo License

File

PLANMECCA Romexis

All Patients | DICOM Worklist | DICOM Query/Retrieve | Find by Image | Cloud Management

Find by Name

Person ID	Last Name	First Name	Date Of Birth	Age	Gender	Phone Number
Planmecca2	Patient	Test				
Planmecca4	Monitor	Test				
LK234	Kobold	Lynda	09/05/14	0		
MB112	Brown	Mary				

Patients

Files

Imaging

3D

CAD/CAM

Clinic

Report

PLANMECCA
Romexis



Choose your files

- Select CAD/CAM from the left hand menu
- Select the files you want to transfer
- Click the “DDX Export” link

Case Name	Image Date	Last Updated
Crown 1-6, Crown 1-3	23/10/14 7:04 AM	23/10/14 7:04 AM
[8-11-2014 12.29.23 PM]	11/08/14 12:33 PM	11/08/14 12:33 PM
Crown 1-4	28/05/14 4:17 PM	12/06/14 3:59 PM
Crown 1-6, Crown 1-4, Crown 1-3, Crown 1-2, Crown 1-1	28/05/14 4:10 PM	14/01/15 1:34 PM
Crown 1-8, Crown 1-6, Crown 1-4, Crown 1-1, Crown 2-4, Crown 2-6	14/05/14 11:44 AM	20/01/15 3:55 PM



Create DDX Case

- Click “Create A New Case”
- Complete the DDX Case in the web browser that opens
- Leave DDX Cloud window open in the background during case creation

DDX Cloud

Existing Cases

Case id	Status	Lab Name	Add Date	Upd Date	Cancel date	Cance...	Patient id	Practi...	Patient Name
114	OPEN	HS Dental Lab	2014-05-23...	2014-05-28...	LK234	2	Kobold, Lynda
119	CLOS...	HS Dental Lab	2014-05-28...	2014-06-05...	LK234	2	Kobold, Lynda
1318	OPEN		2014-09-26...	2014-09-26...	LK234	3	Kobold, Lynda
1376	CLOS...	HS Demo Lab	2014-05-14...	2014-05-23...	LK234	2	Kobold, Lynda

Selected Case

Case Info

Case Files

Image Name	Created	Mime Type	Originator	Originator Name
------------	---------	-----------	------------	-----------------

Shared Accounts

Patient LK234 Kobold Lynda

Image Type	Size (KB)	Exposure Date	Updated date	Comment
Surf	18305.0	20/01/15	20/01/15	Prep
Surf	23522.0	20/01/15	20/01/15	Opposing

Get New Case Info **Create A New Case** Add File(s)

STL (3D Models Only) PLY (3D Models + ICE View) Native PlanCAD (All Data)

Log List

Close



Select A Lab


- Select a lab from the list in My Labs and click “Use Lab”

Schedule a New Case

Select a Lab

My Labs <div style="border: 1px solid #ccc; padding: 5px;"><ul style="list-style-type: none">DDX Dental LabWill's Dental LabHS Demo LabDDX Milling CenterDDX Offline LabJohnsonTeeth 'n StuffHS Dental LabBez LabsBuchanan Laboratory Fake</div>	Find A Lab Search in <input type="text" value="Canada"/> by <input type="text" value="Lab M"/>
--	--

Lab Details

 HS Dental Lab American Fork, UT 801-847-4625	<input type="button" value="✓ Use Lab"/>
---	--



Schedule a New Case



Patient Details

Provider *

First Name

Last Name *

Gender

Patient Chart

Birth Date



Work Requested

Send Date *

Procedure *

Enclosures



Notes

Subject

B **I** **U**

Coupon Code

Tags

Complete the DDX Case

- Select a procedure and fill out all relevant fields
- Click the submit button



Upload Files

- Once case is submitted Romexis will automatically detect the new case and upload the selected files.
- Uploading progress is indicated by the progress bar at the bottom of the DDX Cloud window highlighted in this picture
- The DDX Cloud window will close automatically when complete

DDX Cloud

Case id	Status	Lab Name	Add Date	Upd Date	Cancel date	Cance...	Patient id	Practi...	Patient Name
438	OPEN	HS Dental Lab	2014-12-22...	LK234	2	Kobold, Lynda
438	OPEN	HS Dental Lab	2014-12-22...	LK234	2	Kobold, Lynda
567	OPEN	HS Dental Lab	2015-02-10...	LK234	2	Kobold, Lynda
95	OPEN	HS Dental Lab	2014-05-14	2014-05-15	LK234	2	Kobold, Lynda

Selected Case: Case# 567 of patient Kobold, Lynda

Case Info

Image Name	Created	Mime Type	Originator	Originator Name
------------	---------	-----------	------------	-----------------

Shared Accounts

Patient: LK234 Kobold Lynda

Image Type	Size (KB)	Exposure Date	Updated date	Comment
Surf	18305.0	20/01/15	20/01/15	Prep
Surf	23522.0	20/01/15	20/01/15	Opposing

0%

Get New Case Info Create A New Case Add File(s)

STL (3D Models Only) PLY (3D Models + ICE View) Native PlanCAD (All Data)

Log List

Close Cancel



Verify Files are Attached

You can refresh the web browser window to see that the files were successfully attached to the DDX case

Lab Cases Finances

Case #567 Print Cancel

Return Date:
Feb 12, 2015

Send Date:
Feb 10, 2015

Provider:
Dr. Nick DeSalvo

Case Monitoring:
Not Following

Patient:
[Lynda Kobold](#)

Patient Appointment:
Not Entered

Status:
DDX Case in Progress

Practice Arrived:
Not Arrived

Tags:
None

Details Activity Log (3)

Work Requested

Units	Description
1	e.max Crown [+]

Attached Files

File: Prep_2.16.840.1.113 669.632.10.2015012 0.155533515.45.2.stl Date: Feb 10, 2015 From: Practice	File: Opposing_2.16.840. 1.113669.632.10.201 50120.155530083.43 .1.stl Date: Feb 10, 2015 From: Practice



If you have any questions or would like to schedule a web-demo of DDX please contact:

Michael LeBlanc

Michael.leblanc@henryschein.com

1-877-244-9345